



POINT-OF-CARE



OSOM® Ultra Flu A&B Test

FOR THE DETECTION OF INFLUENZA TYPE A AND TYPE B NUCLEOPROTEIN ANTIGENS

Features and Benefits

- **Accurate** – Test and treat with confidence
- **CLIA-waived** – Easy to use. No need for reader
- **Flexible** – Approved for multiple viral transport media types
- **Ready-to-use** – Collection Swabs and Controls Included
- **Exceptional Customer Service** – Online training modules and technical support staffed by certified medical technologists/professionals

Performance Characteristics

	Sensitivity*	Specificity*	Sample Type	Limit of Detection**	Storage Temperature	Time to Result
Flu A	91.4%	95.7%	Nasal/nasopharyngeal/ nasal wash	9.95 x10 ¹ TCID ₅₀ /mL	2-30°C/35-86°F	10-15 minutes
Flu B	87.6%	95.9%				


*Combined prospective clinical RT-PCR comparator study. See IFU

**LOD for Influenza A/Victoria/3/75 (H3N2) Refer to IFU for additional LODs




OSOM® Ultra Flu A&B Test

SIMPLE TO USE / RESULTS IN MINUTES

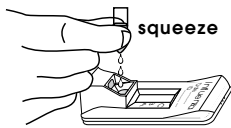


1 DISPENSE ENTIRE CONTENTS OF REAGENT CAPSULE INTO EXTRACTION WELL.

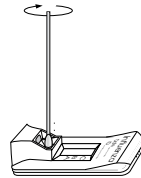

2 PLACE PATIENT SWAB IN SWAB STAND. SPIN SWAB 3 TIMES BEFORE AND **AFTER** 1 MINUTE INCUBATION.




3 DISCARD SWAB. RAISE DEVICE. LET STAND 1-2 SECONDS.

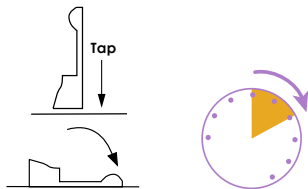

4 GENTLY TAP TO ENSURE LIQUID FLOWS INTO HOLE OF DEVICE. LAY THE DEVICE BACK DOWN. START TIMING. READ RESULTS AT 10-15 MINUTES. CONFIRM NEGATIVE RESULTS AT 15 MINUTES.


POSITIVE
Any visible A and/or B Line(s)

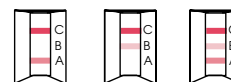

NEGATIVE
C Line Only



INVALID
NO C Line

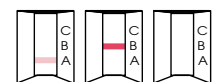

Ordering Information

	Configuration	Catalog Number	Storage	CPT Code
OSOM ULTRA FLU A&B TEST	25 Tests	1006	Room Temp	FLU A - 8704QW FLU B - 87804-59
OSOM ULTRA FLU A & B CONTROL KIT	5 X Pos Flu A/B Swabs 5 X Neg Flu A/B Swabs	1005	Room Temp	
OSOM ULTRA FLU A & B NASOPHARYNGEAL WASH/ASPIRATE KIT	50 Pipettes; 5ml Extraction Reagent	1007	Room Temp	

Additional Accessories (Swabs and Swab With Transport Tubes) are available.

To learn more about all of our OSOM Tests scan the code or go to osomtests.com



THE AMERICAS

SEKISUI Diagnostics, LLC
One Wall Street
Burlington, MA 01803
Phone: 800 332 1042
Fax: 800 762 6311

Email: questions@sekisui-dx.com

SEKISUI
DIAGNOSTICS

Because every result matters™

sekisuidiagnostics.com

